

MALEM MEDICAL

Queen's Square[®] Bladder Stimulator

Vibrating Bladder Emptying Device



Hand Held - Battery Operated

The Stimulator works on two AA Alkaline batteries (not included)

Instruction for use:

This product can be used to initiate urination and bladder emptying. Adjust cord length by tying a knot. Secure stimulator by placing your hand through the attached cord.

1. Place the Stimulator on the lower abdomen (above the pubic bone) before commencing passing urine and apply gentle pressure to activate the vibration. (If you are unable to exert sufficient pressure to activate, then use the on/off button on the back.)
2. Keep the stimulator vibrating while urine is being passed and for about one to two (1-2) minutes after you feel you have finished.

If you wish further clinical information, please contact: Clinical Nurse Specialist in Uro-Neurology
The National Hospital for Neurology and Neurosurgery, Queen Square, London WC1 3BG

Reference: British Journal of Urology, Volume 80, Issue 2, August 1997. P234-237



Malem Medical, 10 Willow Holt, Lowdham, Nottingham NG14 7EJ UK
Tel: ++ 44 (0) 115 966 4440 Fax: ++ 44 (0) 115 966 4672
E-mail: malem@malem.co.uk www.malem.co.uk

